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Thí dụ Home Monitoring

BIOTRONIK Home Monitoring Service Center 3.0

Dr. Marcus Welby August 12, 2007 12:17

Monitoring

- Patients for review
- All patients
- New patient

Administration

- Users
- New user
- Patient groups
- New patient group
- Transmitters
- Option templates

Site Tools

- Home
- What's new
- User profile
- Contact
- Imprint
- Help
- Sign Out

Your patients today

You've got **14 patients** for review.

Out of **21 patients** with activated monitoring,

- 4 have a **red** status.
- 10 have a **yellow** status.

Important information!

This internet platform is a Demonstrator of the HMSC 3. Therefore only some of the functions will be provided. Furthermore the provided functions can be limited. For information about what functions will be provided by this Demonstrator read the demonstrator guide.

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Plot Summary for
"Marcus Welby, M.D." (1969) [More at IMDbPro](#) »

The show is about doctors Marcus Welby, a general practitioner and Steven Kiley, Welby's young assistant. The two try to treat people as individuals in an age of specialized medicine and uncaring doctors. *Written by Anonymous*

Thí dụ Home Monitoring

BIOTRONIK

Home Monitoring Service Center 3.0

?

Monitoring

Patients for review

All patients

New patient

Administration

Users

New user

Patient groups

New patient group

Transmitters

Option templates

Site Tools

Home

What's new

User profile

Contact

Imprint

Help

Sign Out

↑

Back to Top

Patients for review

Patients 1-10 of 14

Search

Display

10

20

50

1-10 of 14

Patient ID	Finding	Implant type/SN	Implantation	Comment
pat_111 Kate Bloe	Implant Backup,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
pat_44 James Doe	Ven. arrhythmia VF,...	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
pat_444 Walt Doe	Ven. arrhythmia VF,...	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
patient_1009 Mary Major	Ven. arrhythmia VF,...	Lumax 300 HF-T SN:55550000	Oct 1, 2007	Medical history : pre-implant successful atrial ablation (patient presented with perm. Afi...
pat_01 John Jack Doe	Leads LeadLV,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
pat_02 Charly Roe	HF Monitor ABurd,...	Evia DR-T SN:114422	Dec 10, 2009	persistent AF; 2002 DDD implant due to AV III*
pat_03 Libby Roe	HF Monitor ABurd,...	Evia DR-T SN:3288	Feb 11, 2010	SSS; symptomatic AF tachycardia
pat_04 Jacob Doe	Holter / Episode P-IEGM	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
pat_05 Michael Doe	HF Monitor PVC	Lumax 300 HF-T SN:231300	Oct 31, 2009	
pat_1 Walt Doe	Leads LVLead,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	
pat_11 Claire Roe	Leads LeadLV,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
patient_1001 John Stiles	Leads LeadRV	Lumax 300 VR-T SN:99990000	Nov 5, 2007	Medical history: ICM (EF 38%) - CAD stage 3 - inferior wall infarct 2003 - RCA-PCI 2003 - ...
patient_1005 J. J. Roe	Leads LeadRV	Cylos DR-T SN:12340000	Jul 19, 2005	
patient_1007 John Doe	Bradycardia/CRT LowCRT	Lumax 300 HF-T SN:11110000	Jun 30, 2007	Medical history: ICM mit 23% EF - RCA, previous stenosis; Current status: AV-Block III* wi...

View

Patients for review

All patient groups

All implant types

Activated monitoring

✓

Extended view

Save/print (PDF)

T.s. Trần Trọng

VNSEP 07/2011

Thí dụ Home Monitoring

BIOTRONIK Home Monitoring Service Center 3.0

Monitoring

- Patients for review
- All patients
- New patient

Administration

- Users
- New user
- Patient groups
- New patient group
- Transmitters
- Option templates

Site Tools

- Home
- What's new
- User profile
- Contact
- Imprint
- Help
- Sign Out

pat_44 James Doe, DOB Apr 30, 1948

Status on Dec 23, 2009 12:19 PM Lumax 540 DR-T / SN:221100
Implantation: Sep 10, 2009

Status Device settings Recordings History Patient profile Options Save/print (PDF)

Summary Device Leads Bradycardia/CRT Atrial arrhythmia Ven. arrhythmia Physiologic. param. HF Monitor

	Ven. arrhythmia	VF detected 3 VF detected between Dec 18, 2009 12:55:36 AM and Dec 23, 2009 12:18:19 PM	New.	☒ Acknowledge
	Holter / Episode	Multiple periodic IEGMs received 2 periodic IEGMs were recorded between Sep 17, 2009 12:40:42 AM and Dec 18, 2009 12:40:42 AM	New.	☒ Acknowledge
		Episode details received Episode details were received for 2 spontaneous VF episode, which were detected between Dec 18, 2009 6:56:25 PM and Dec 23, 2009 12:18:02 PM	New.	☒ Acknowledge
Automatic remark		none		
Show timeline				
Status comment: Add comment				

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Physiologic. param.

HF Monitor

Last device message created on Dec 23, 2009 12:19 PM

VF detected

3 VF detected between Dec 18, 2009 12:55:36 AM and Dec 23, 2009 12:18:19 PM

New.


Acknowledge

Ven. detection	Since Oct 26, 2009 11:53:36 PM	Since implantation
VT1 episodes	0	0
VT2 episodes	0	0
VF episodes	3	3
Episodes during DTM	0	1
SVT episodes (total)	0	0
Ven. ATP		
ATP in VT zones started	0	0
ATP in VT zones successful	0	0
ATP One Shot started	0	0
ATP One Shot successful	0	0
Ven. shocks		
Shocks started	3	3
Shocks aborted	2	2
Shocks successful	1	1
Ineffective ven. max. energy shocks	0	0
PVC	24 h	Since Oct 26, 2009 11:53:36 PM
Mean PVC/h	26	Mean values
Last episode		
No.	6	
Type	VF	
Detection	Dec 23, 2009 12:18:02 PM	


Có biểu đồ

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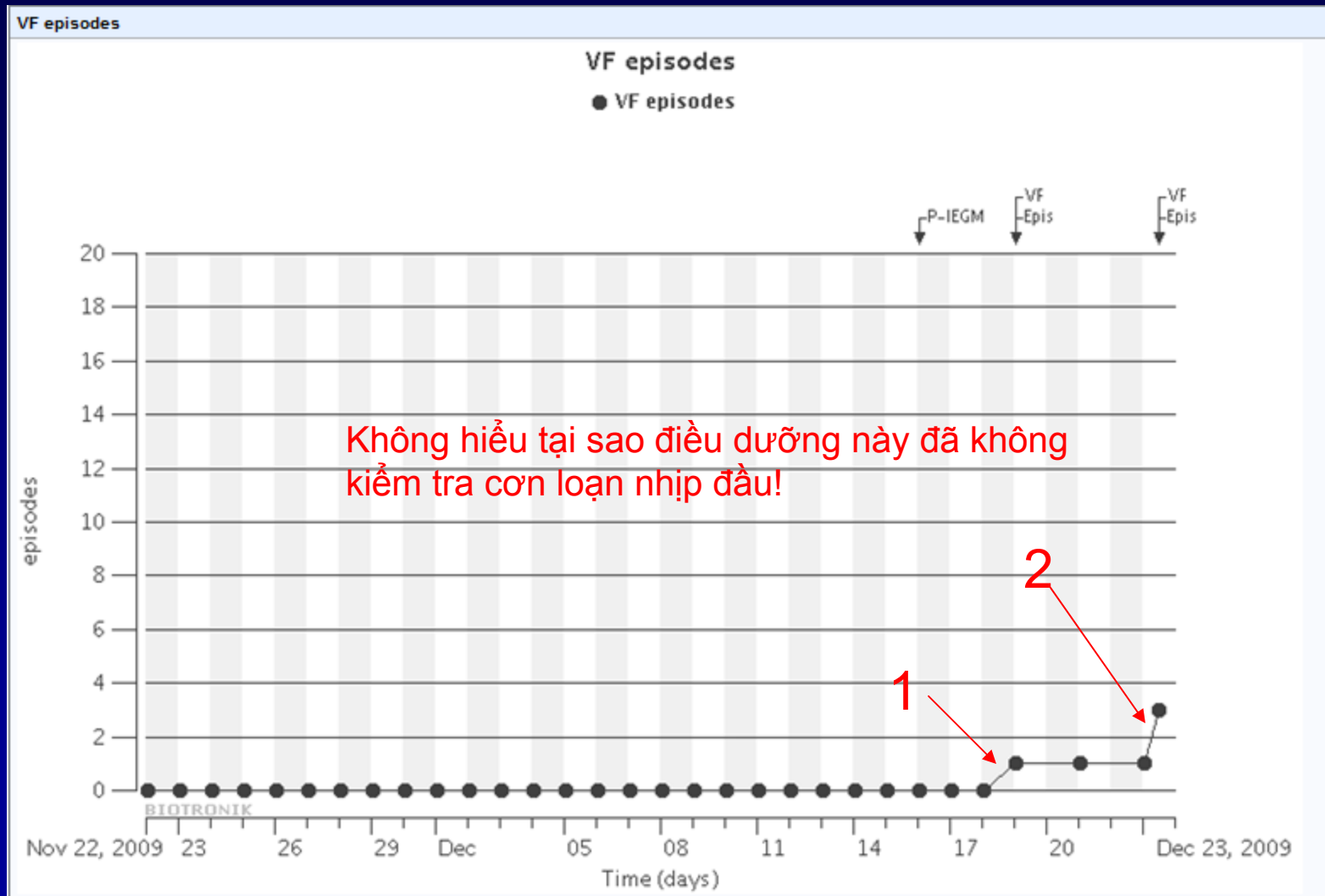Thí dụ Home Monitoring

Thí dụ Home Monitoring

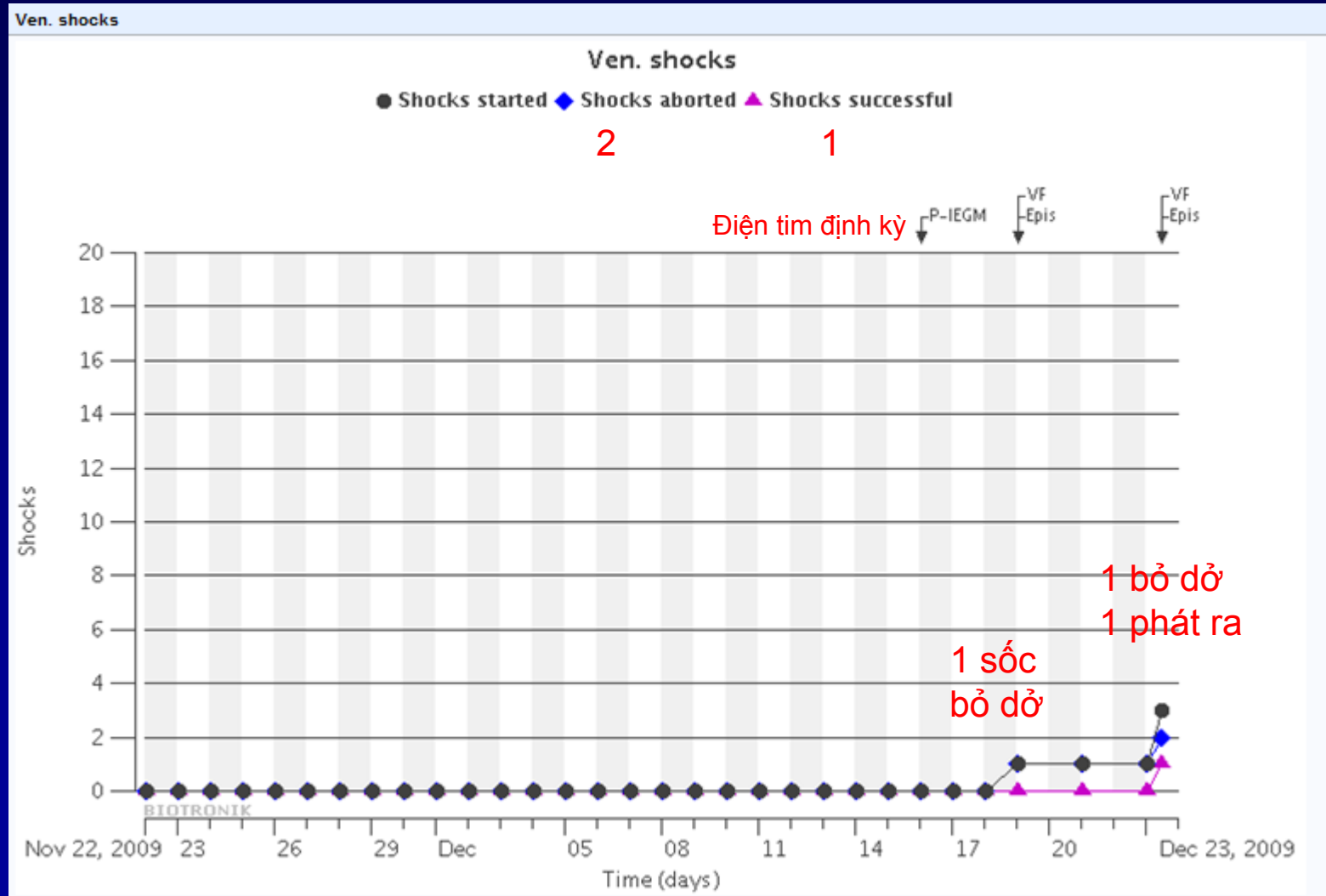Thí dụ Home Monitoring

BIOTRONIK Home Monitoring Service Center 3.0

pat_44 James Doe, DOB Apr 30, 1948 Patients (filtered) (2/14)

Status on Dec 23, 2009 12:19 PM Lumax 540 DR-T / SN:221100
Implantation: Sep 10, 2009

[Status](#)
[Device settings](#)
[Recordings](#)
[History](#)
[Patient profile](#)
[Options](#)
[Save/print \(PDF\)](#)

Display 10 20 50 1-6 of 6



No.	Time	Type	Details
6	Dec 23, 2009 12:18:02 PM	VF	Shocks delivered: 1
4	Dec 18, 2009 6:56:25 PM	VF	Shocks aborted: 1
3	Dec 16, 2009 12:40:42 AM	Periodic IEGM	Monitoring only
	Oct 26, 2009 10:26:07 AM	Follow-up	
2	Sep 17, 2009 12:40:42 AM	Periodic IEGM	Monitoring only
	Sep 10, 2009 11:22:25 AM	Follow-up	

View
☒ Periodic IEGMs
 ☒ Supraventr. episodes
 ☒ Ven. episodes
 ☐ Since last follow-up
 ☒
[Save/print \(PDF\)](#)


Note: A red arrow points from the Vietnamese text below to the entry with No. 6 in the table.

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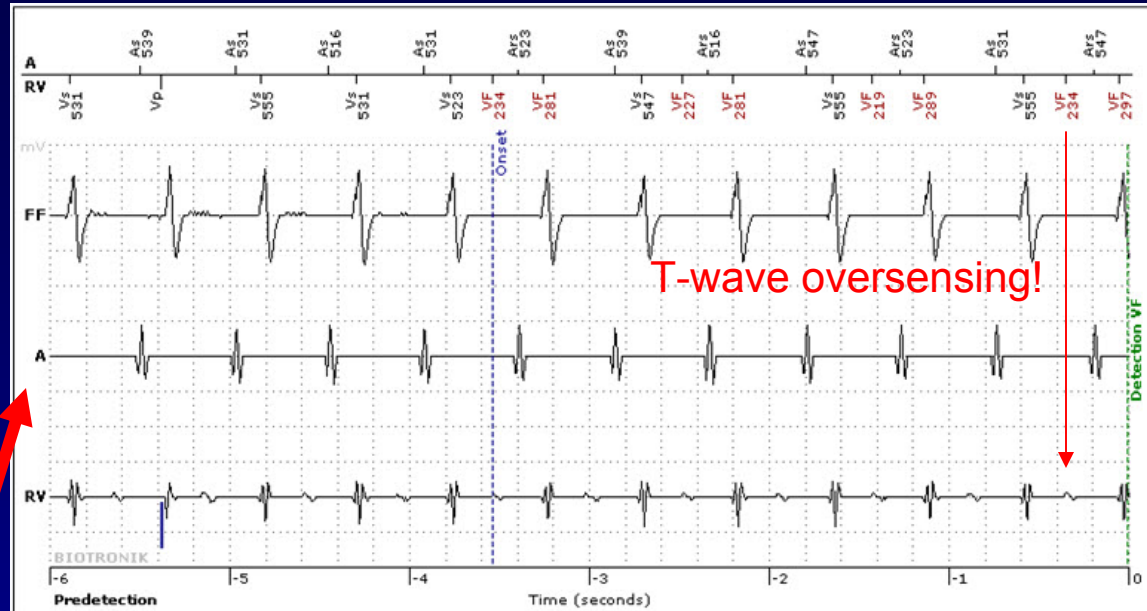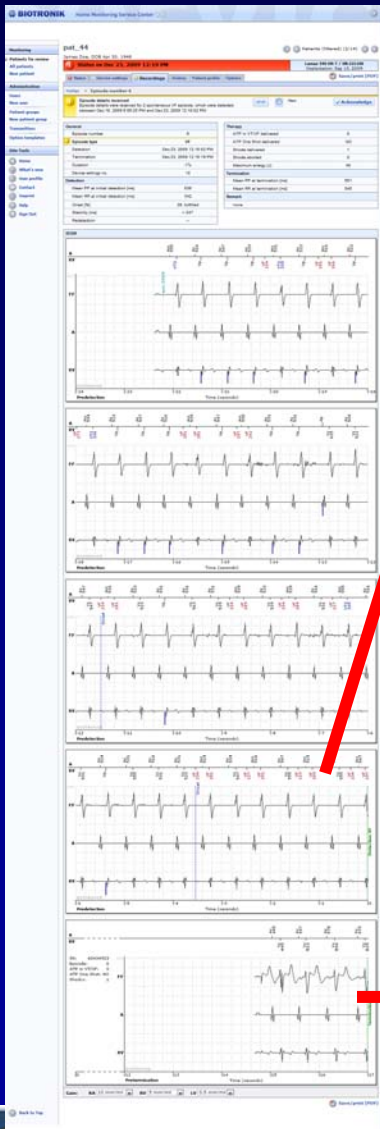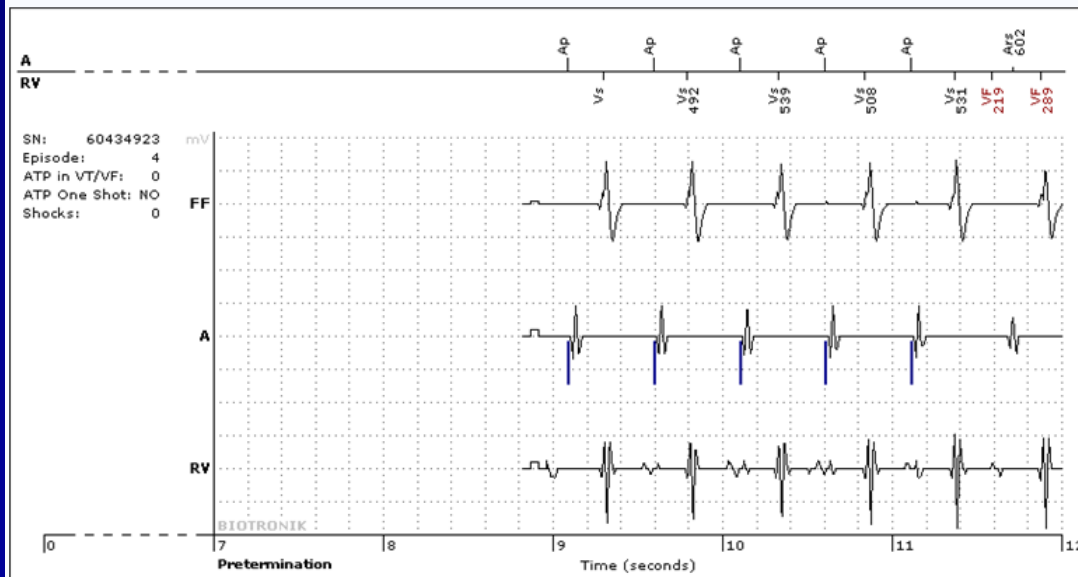
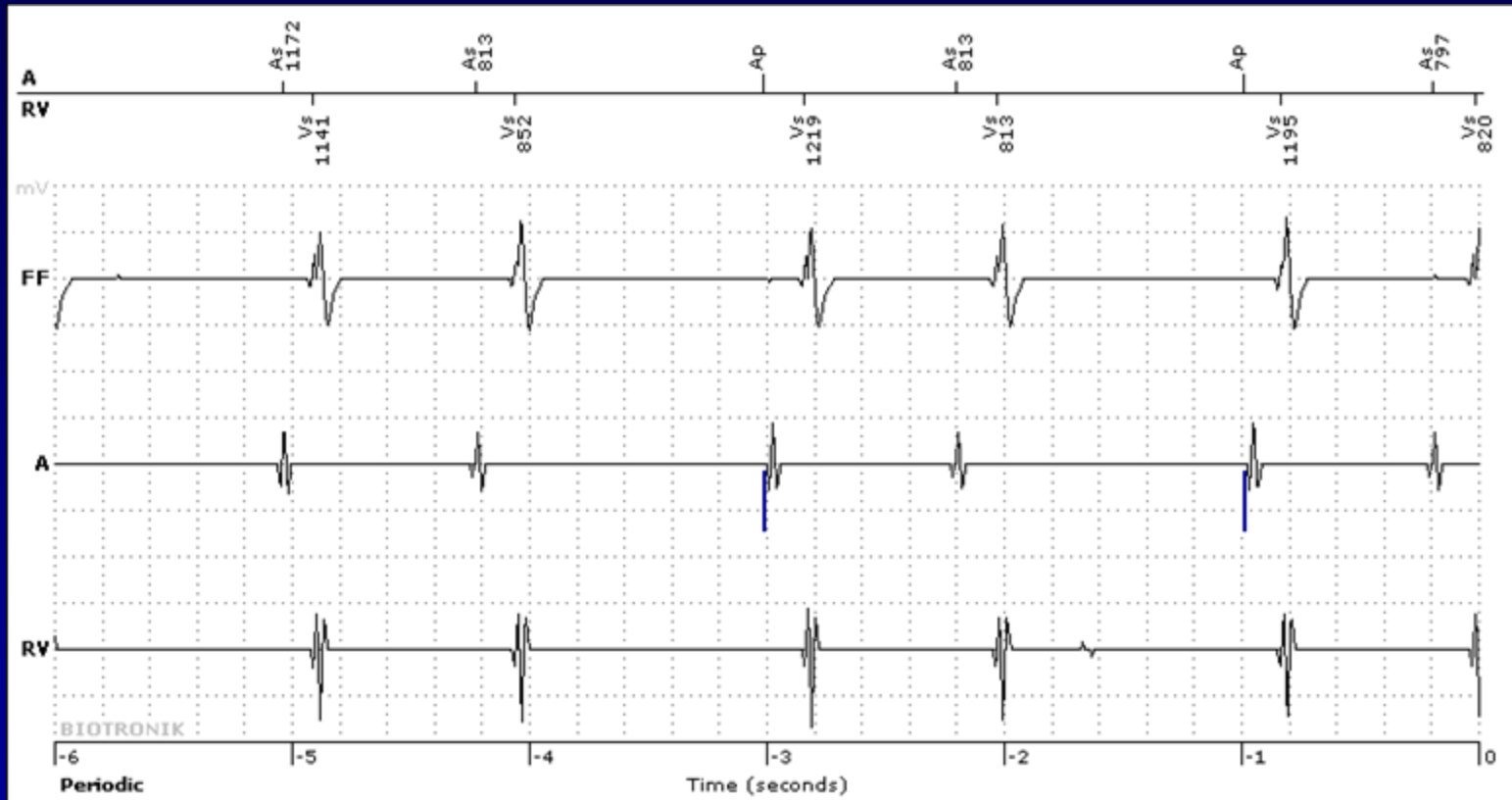
Save/print (PDF)

Summary Device Leads Bradycardia/CRT Atrial arrhythmia Ven. arrhythmia Physiologic. param. HF Monitor

Last device message created on Sep 8, 2006 03:23 AM

Battery	
Battery status	BOL
Battery voltage [V]	3.12
Date of last battery voltage measurement	Sep 08, 2006
Implant	
Implant status	OK
Last entry of shock holler	
Time	Sep 08, 2006 04:12 PM
Charge time [s]	7.9
Energy [J]	30
Impedance [ohm]	50
Remark	--
Home Monitoring	
Type of message	event triggered
Message created on	Sep 08, 2006 5:19:16 PM
End of last monitoring interval	Sep 08, 2006 12:55:36 AM
Transmitter SN	46809898

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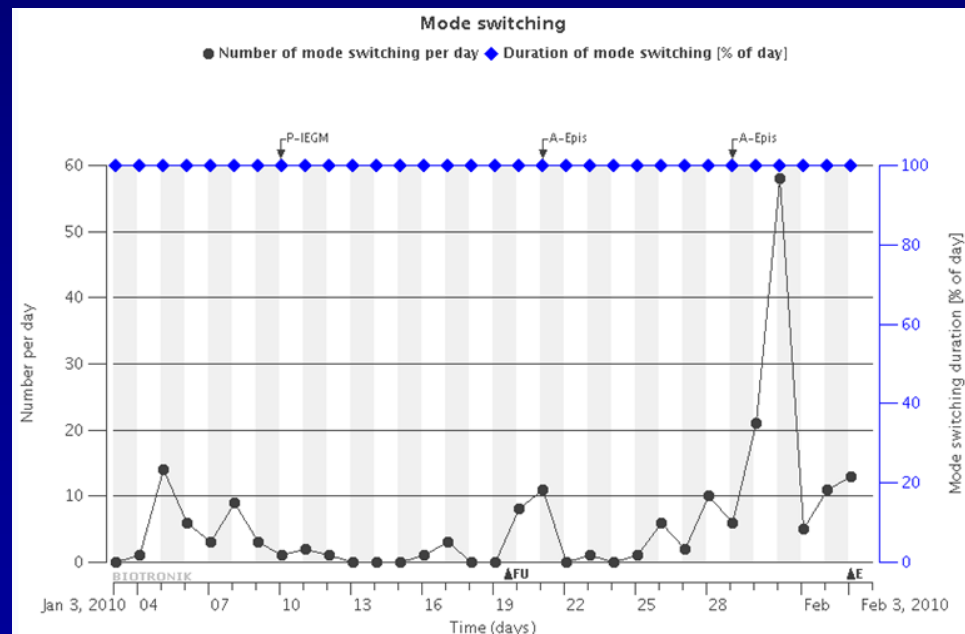
Patients for review				
Patients 1-10 of 14				
Search		Display 10 20 50		
Patient ID	Finding	Implant type/SN	Implantation	Comment
pat_111 Kate Bloe	Implant Backup,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
pat_44 James Doe	Ven. arrhythmia VF,...	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
pat_444 Walt Doe	Ven. arrhythmia VF,...	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
patient_1009 Mary Major	Ven. arrhythmia VF,...	Lumax 300 HF-T SN:5550000	Oct 1, 2007	Medical history : pre-implant successful atrial ablation (patient presented with perm. AFi...
pat_01 John Jack Doe	Leads LeadLV,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
pat_02 Charly Roe	HF Monitor ABurd,...	Evia DR-T SN:114422	Dec 10, 2009	persistent AF; 2002 DDD implant due to AV III*
pat_03 Libby Roe	HF Monitor ABurd,...	Evia DR-T SN:3288	Feb 11, 2010	SSS; symptomatic AF tachycardia
pat_04 Jacob Doe	Holter / Episode P-JEGM	Lumax 540 DR-T SN:221100	Sep 10, 2009	Cardiomyopathy, EF 37%, Brady-arrhythmia; renal failure
pat_05 Michael Doe	HF Monitor PVC	Lumax 300 HF-T SN:231300	Oct 31, 2009	
pat_1 Walt Doe	Leads LVLead,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	
pat_11 Claire Roe	Leads LeadLV,...	Lumax 540 HF-T SN:221122	Sep 10, 2009	lives 700 km away from ICD clinic; Coronary Artery disease since 2005, bypass x 4 2006, HF...
patient_1001 John Stiles	Leads LeadRV	Lumax 300 VR-T SN:9990000	Nov 5, 2007	Medical history: ICM (EF 38%) - CAD stage 3 - inferior wall infarct 2003 - RCA-PCI 2003 - ...
patient_1005 J. J. Roe	Leads LeadRV	Cylos DR-T SN:12340000	Jul 19, 2005	
patient_1007 John Doe	Bradycardia/CRT LowCRT	Lumax 300 HF-T SN:11110000	Jun 30, 2007	Medical history: ICM mit 23% EF - RCA, previous stenosis; Current status: AV-Block III* wi...
View Patients for review All patient groups All implant types Activated monitoring Extended view				

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


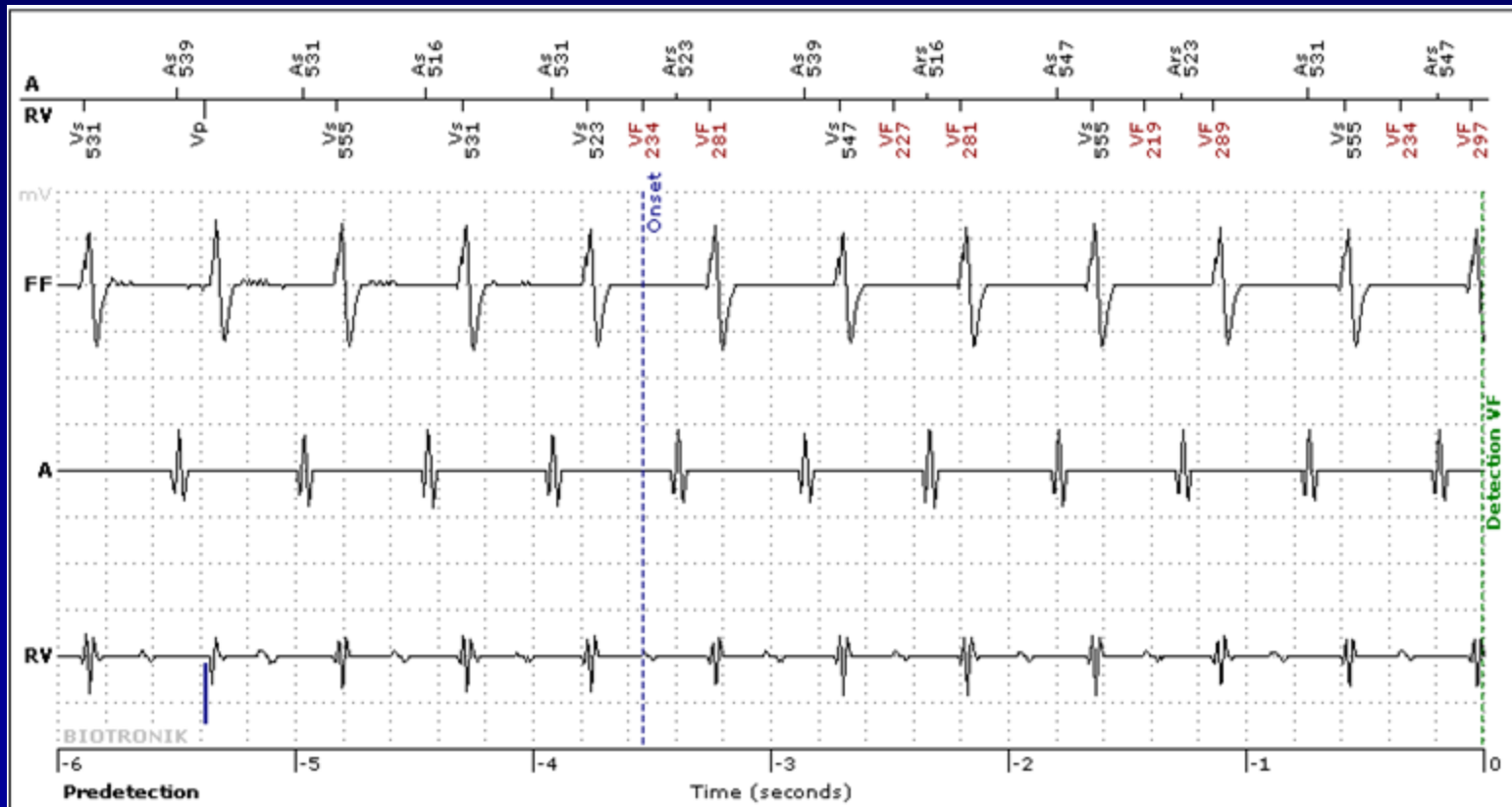
- T-wave oversensing: dùng Select Sense, bật enhanced T-wave suppression

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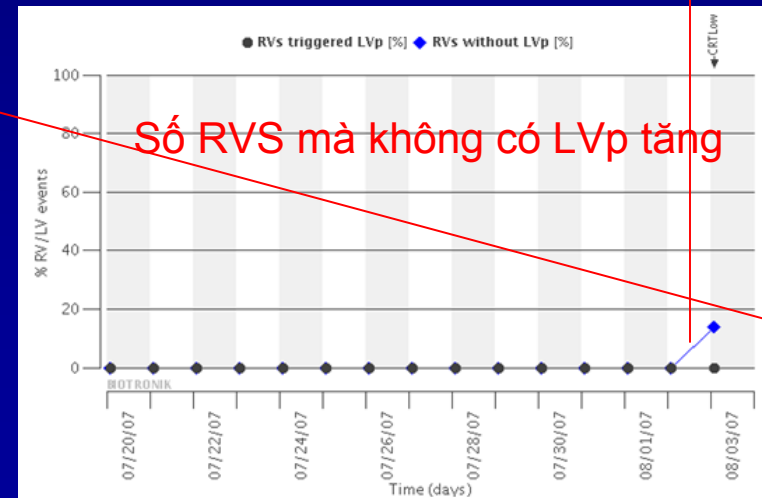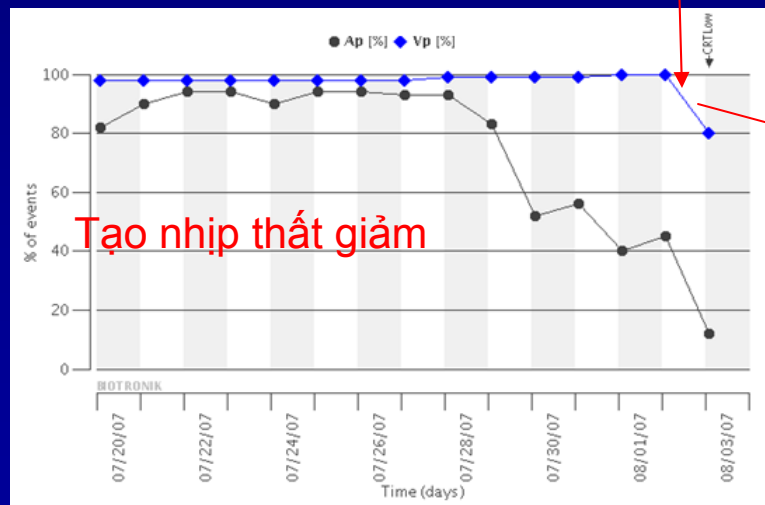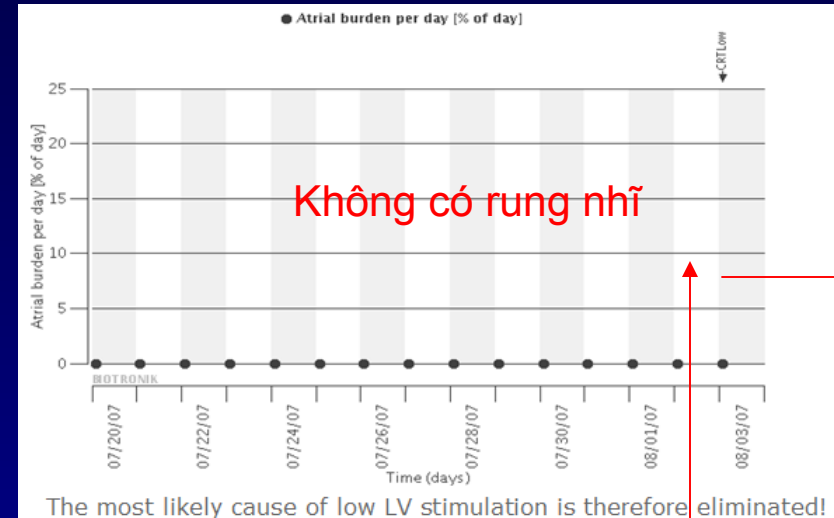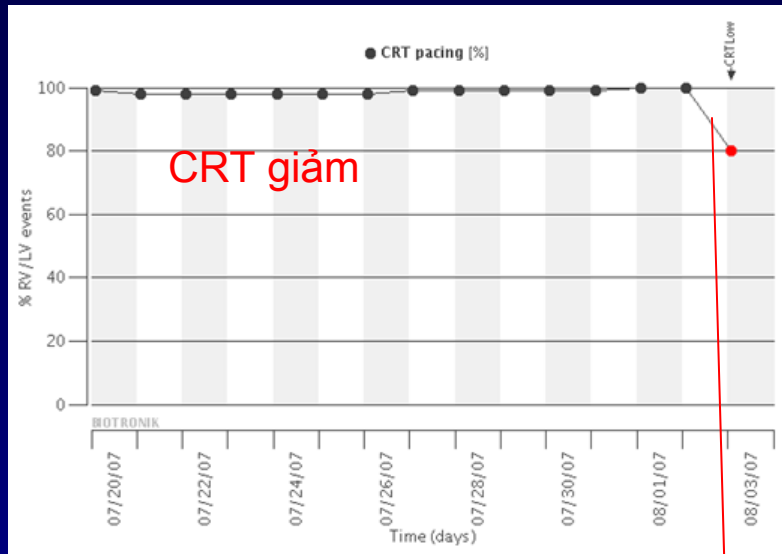
Dramatic increase in PVCs

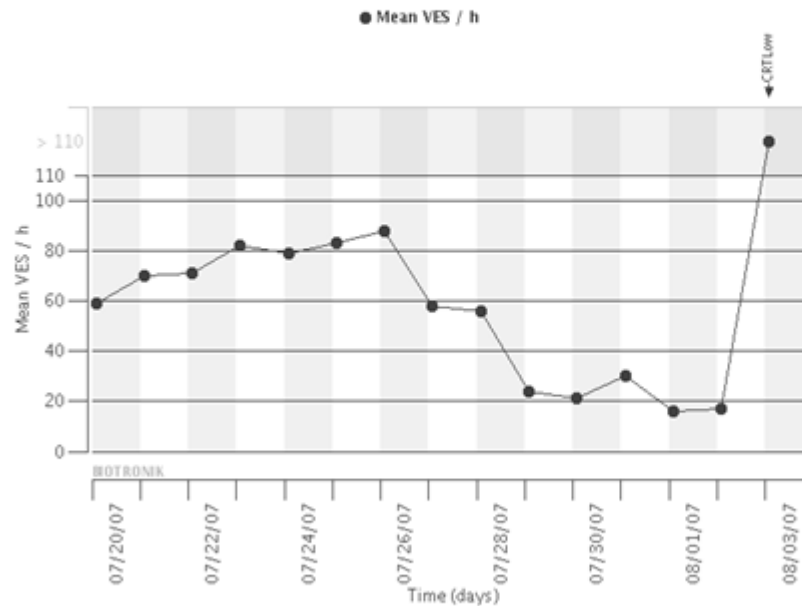Ventricular extrasystoles (PVC's)


PVC's:
Second most common cause of low LV stimulation!

Reason:
Reduction and/or inadequate β -blocker dose

Immediate action: contacted patient via telephone on Aug 3, 2007 and up-titrated β -blocker dose

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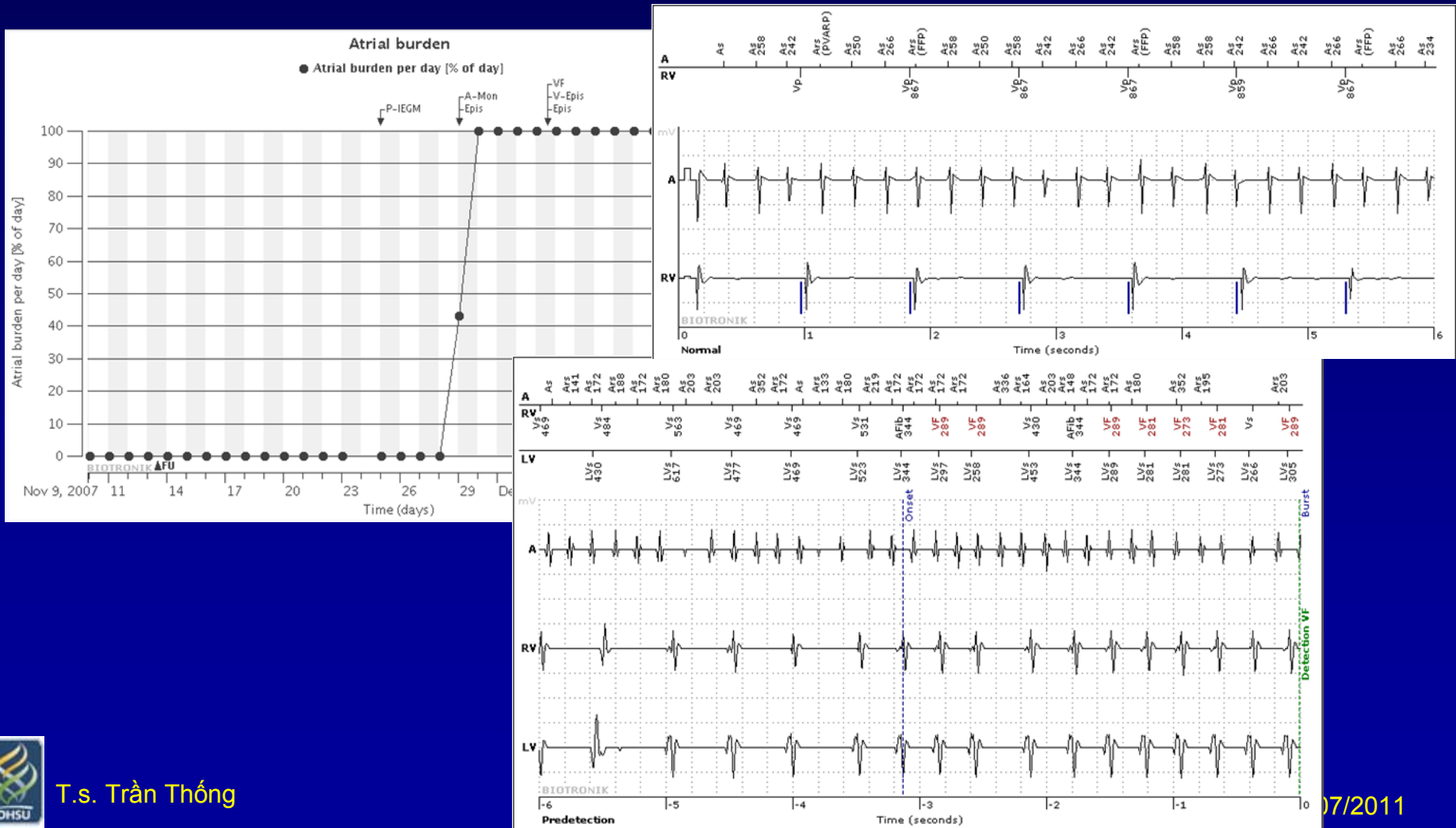
- Giảm CRT

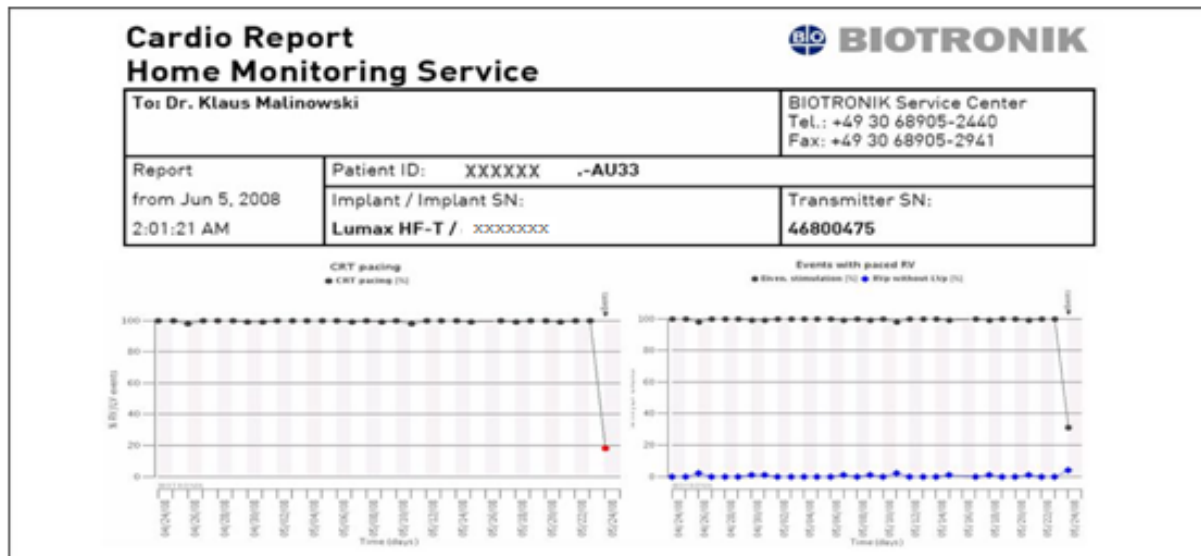Starting on May 24, 2008 Cardio Report showed fall in CRT-Pacing to <85%

- Repeat contact with patient on May 25, 2008
- This time, she complained of a definite "change in rhythm"

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- Called back into the clinic on 26 May 2008
- Patient suffering from slow VT which was slower than the previously-revised VT-zone of 130 bpm
- Successful termination of VT via manual ATP during the visit

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Finally, slow VT-detection zone was reset at 118 bpm and the patient was sent home :

Before		After	
Overview		Detection	
Tachycardia detection : Enabled		Rate	VF
		118	207 bpm
		Detection counter	8 in 12
		Redetection counter	
		20	
		SMART detection	
		ON	
		SMART redetection	
		ON	
		Onset	
		20	
		Stability	
		12	
		Sustained VT	
		12	
		Forced termination	1 min

Rate	1. ATP	2. ATP	Shocks		
			1st	2nd	3-Nth
VT1	130 bpm	3×Burst	OFF		
VT2	167 bpm	3×Burst	30J	30J	6×30J
VF	207 bpm	Ramp	30J	30J	6×30J


Progressive course of therapy		ON			
Mode	DOD				
Ventricular pacing	BiV/T				
Basic rate	60 ppm	Pulse amplitude	2.4	2.4	2.4 V
Upper rate	130 ppm	Pulse width	0.5	0.5	0.5 ms

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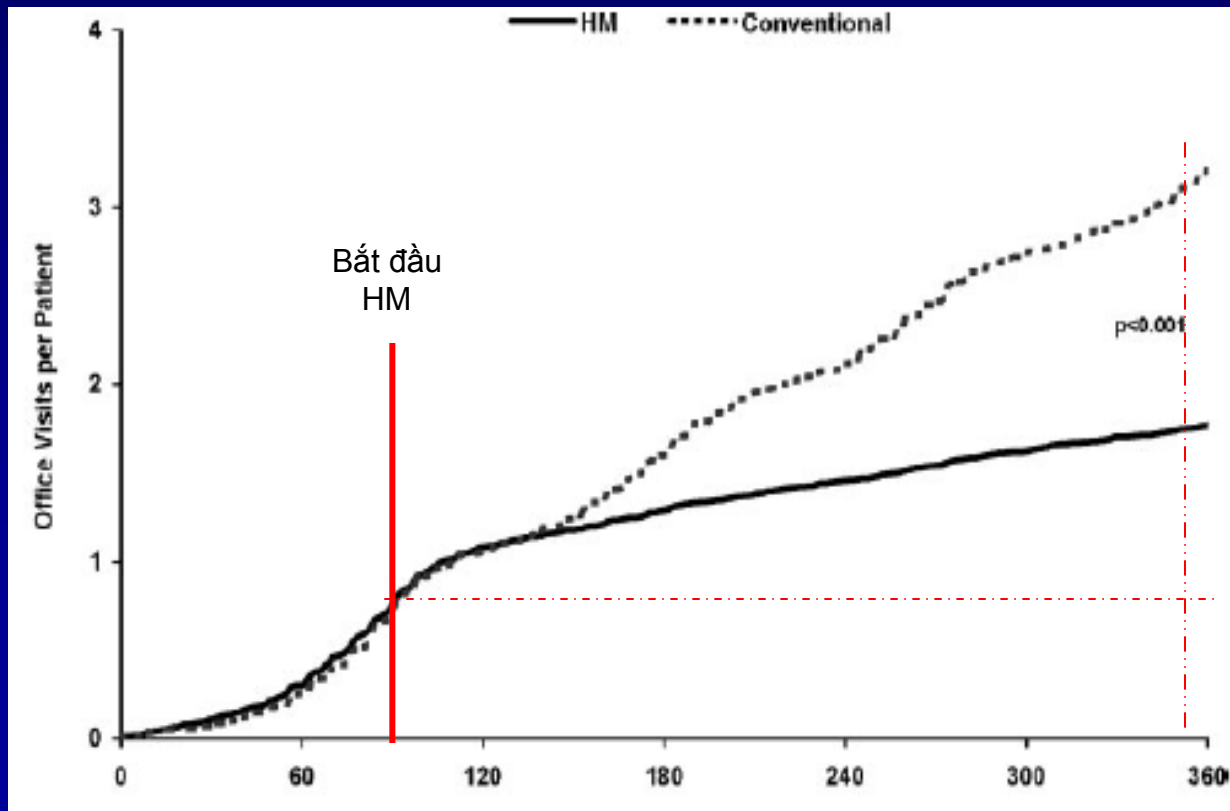
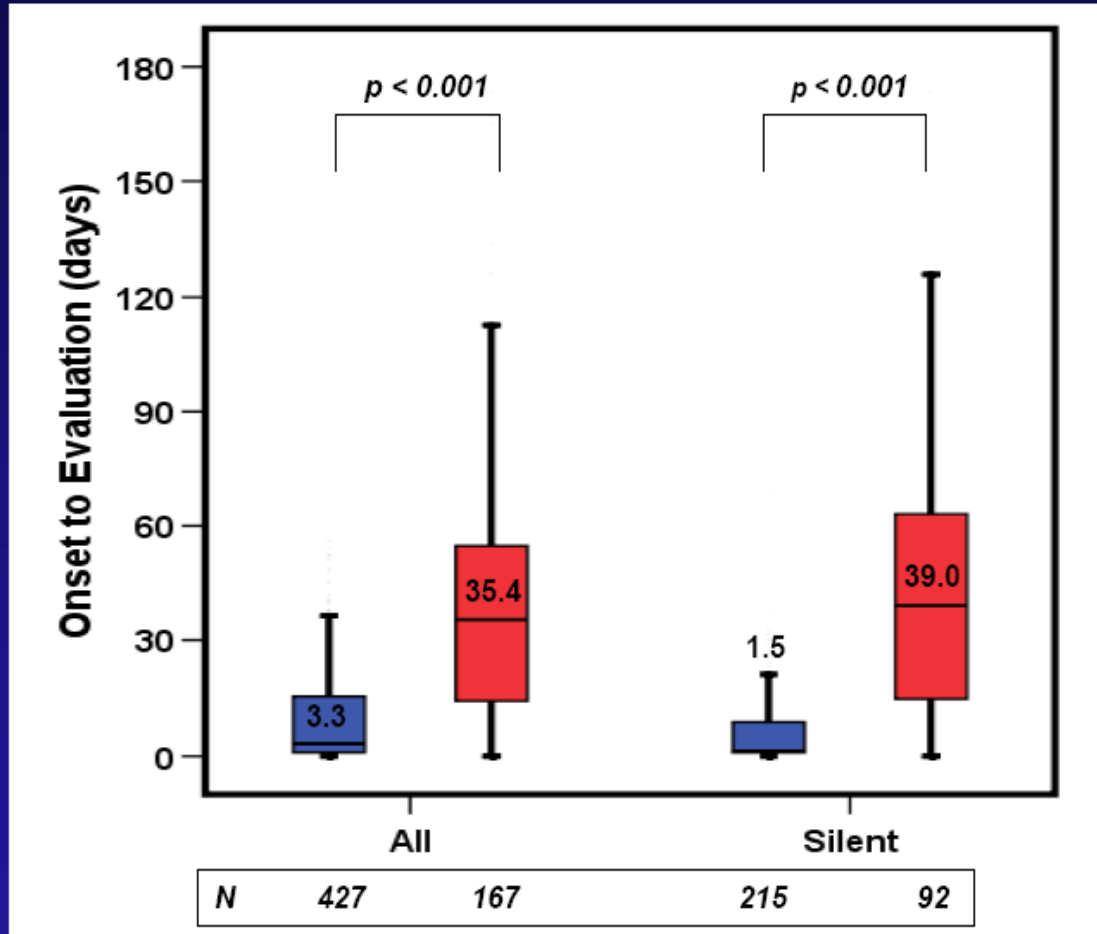
Conventional
Remote
Monitoring

Hiệu quả Home Monitoring

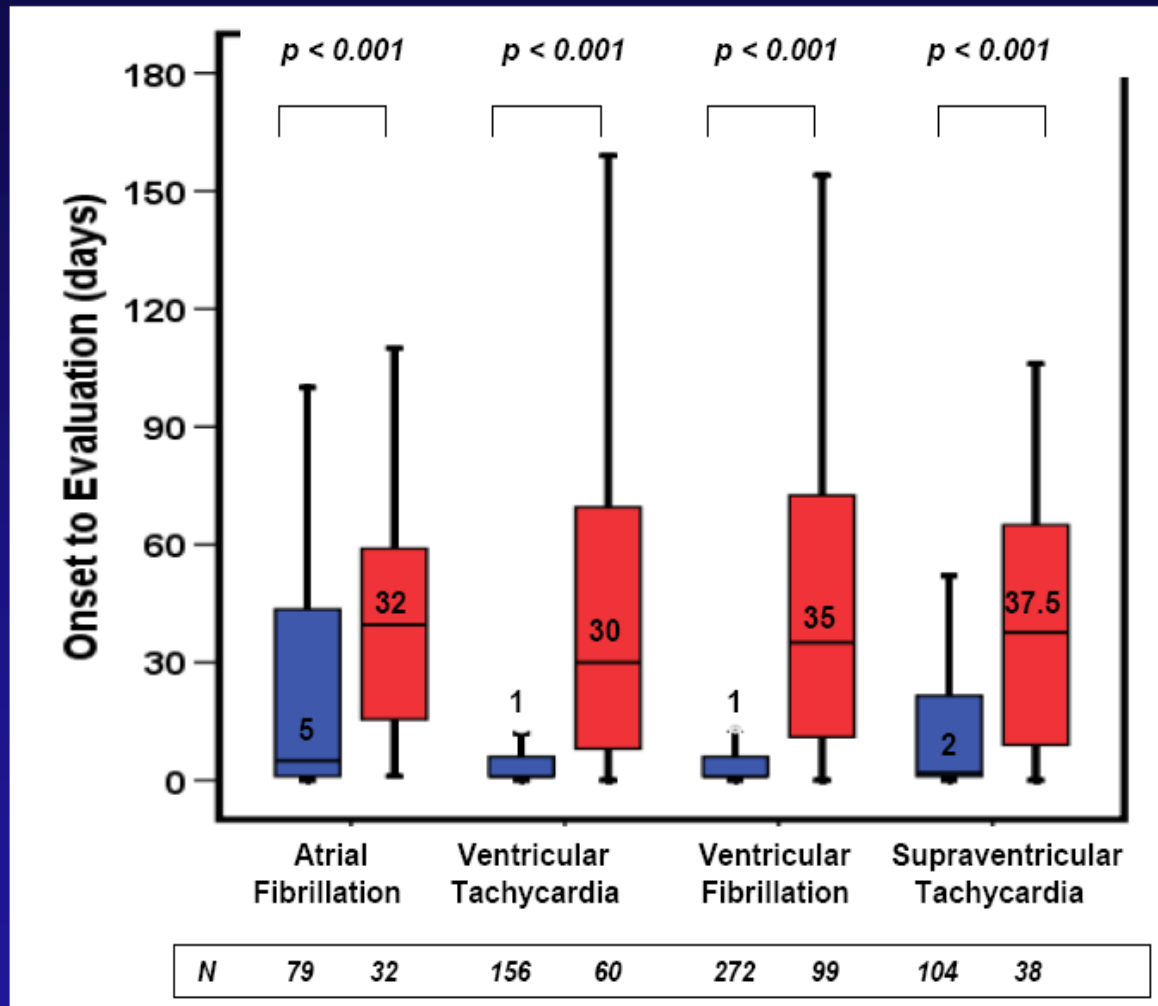Early Event Detection: Arrhythmias

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Home Monitoring VN

Patients 1 - 2 of 2


Search		Display 10 20 50		1 - 2 of 2			
Patient ID	Finding	Device/SN	Implantation	Comment	Patient group	Last message	Last viewed
	No anomalies	Lumax 300 HF-T	Apr 16, 2010	Initial...		May 27, 2011 12:50 AM	May 26, 2011 5:02 AM
	No anomalies	Lumax 300 HF-T	Jun 25, 2010	Male, 1947		May 27, 2011 12:57 AM	May 27, 2011 1:57 AM
View		All patients	All patient groups	All device models	Activated monitoring	✓	Default view

 Save/print (PDF)

Home Monitoring VN

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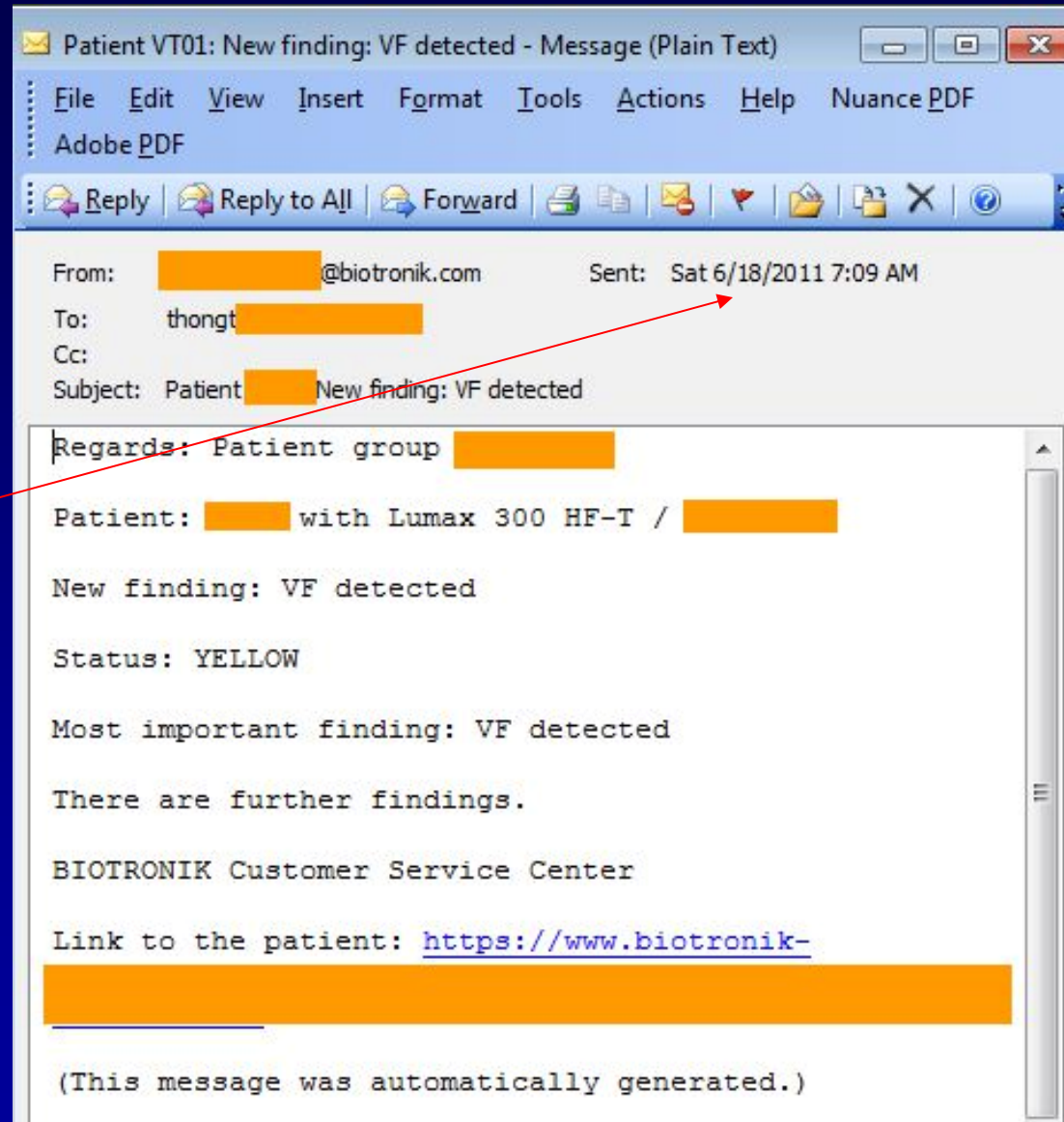Home Monitoring VN

The screenshot shows the BIOTRONIK Home Monitoring Service Center interface. The header includes the BIOTRONIK logo and the text "Home Monitoring Service Center". A sidebar on the left contains navigation links: Monitoring (Patients for review, All patients, New patient), Administration (Users, New user, Patient groups, New patient group, Transmitters, Option templates, Clinic settings), and Site tools (Home, What's new, User profile, Contact, Imprint, Help, Sign out). The main content area is titled "Tran Thong" and shows a date of "Jun 7, 2011 1:42 AM". It features a "Your patients today" section with a progress bar and a message: "You've got 1 patient for review." Below this, it states "Out of 2 patients with activated monitoring, 1 has a yellow status." A "Message of the day" section on the right welcomes the user and lists enhancements as of April 2011: Notifications about automatic deactivation, Enrolling patients, and "Holter" is now "Recordings". It also includes a link to "More news".

BIOTRONIK Home Monitoring Service Center

Tran Thong Jun 7, 2011 1:42 AM

Monitoring

- Patients for review
- All patients
- New patient

Administration

- Users
- New user
- Patient groups
- New patient group
- Transmitters
- Option templates
- Clinic settings

Site tools

- Home
- What's new
- User profile
- Contact
- Imprint
- Help
- Sign out

Your patients today

You've got 1 patient for review.

Out of 2 patients with activated monitoring, 1 has a yellow status.

Message of the day

Welcome,

as of April 2011, the following enhancements in Home Monitoring are available to you:

- Notifications about automatic deactivation
- Enrolling patients
- "Holter" is now "Recordings"

For more details, please follow the link "More news".

[More news](#)

Home Monitoring VN

Last entry of shock list	
Time	Jun 18, 2011 7:58:53 PM
Charge time [s]	3.0
Energy [J]	20
Impedance [ohm]	49
Remark	inverse
Home Monitoring	
Message type	event triggered
Device message created on	Jun 18, 2011 7:59:02 PM
End of last monitoring interval	Jun 18, 2011 12:55:36 AM
Transmitter	
Transmitter SN	46820378
Last transmission received on	Jun 18, 2011 9:05:50 PM

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Home Monitoring VN

The screenshot displays the Biotronik Home Monitoring Service Center interface. The left sidebar contains navigation links for Monitoring, Administration, Users, Patient groups, Transmitters, Option templates, Clinic settings, and Site tools. The main content area shows the patient's status on June 18, 2011, at 9:08 PM. A yellow banner indicates a 'Ven. arrhythmia' episode. The episode details show 'VF detected' between June 9, 2011, 1:00:36 AM and June 18, 2011, 7:59:02 PM. A red arrow points to the time '19:58:48' in the episode details, with the text 'Cơn rung thất 19:58:48' (Ventricular arrhythmia 19:58:48) written in red. The interface also includes buttons for 'Acknowledge', 'Add comment', and 'Show timeline'.

BIOTRONIK Home Monitoring Service Center

Monitoring

Patients for review

All patients

New patient

Administration

Users

New user

Patient groups

New patient group

Transmitters

Option templates

Clinic settings

Site tools

Home

What's new

User profile

Contact

Imprint

Help

Sign out

Back to top

Back to overview

Patient 1 / 2 (filtered)

Status on Jun 18, 2011 9:08 PM

Lumax 300 HF-T / SN: [redacted]

Implantation: Jun 25, 2010

Save/print (PDF)

Status

Device settings

Recordings

History

Patient profile

Options

Summary

Device

Lead

Bradycardia/CRT

Atr. arrhythmia

Ven. arrhythmia

Physiologic. param.

HF monitor

Ven. arrhythmia

VF detected

2 VF detected between Jun 9, 2011 1:00:36 AM and Jun 18, 2011 7:59:02 PM

New.

Acknowledge

Recordings / Episode

Episode details received

Episode details were received for a spontaneous VF episode, which was detected on Jun 18, 2011 7:58:48 PM

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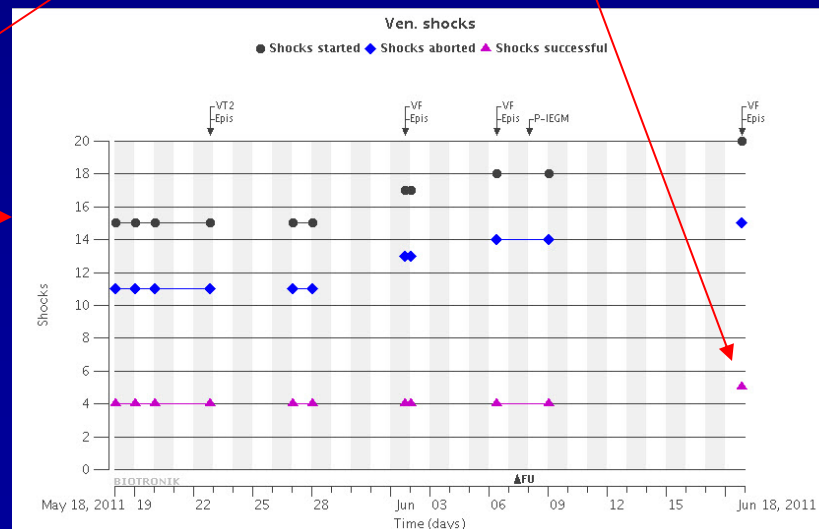
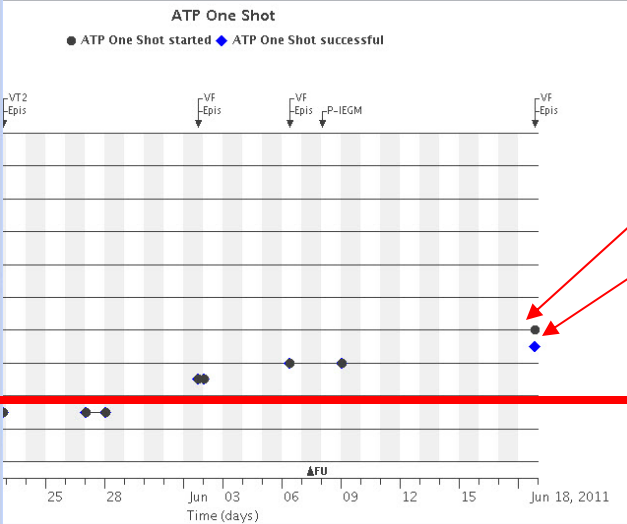
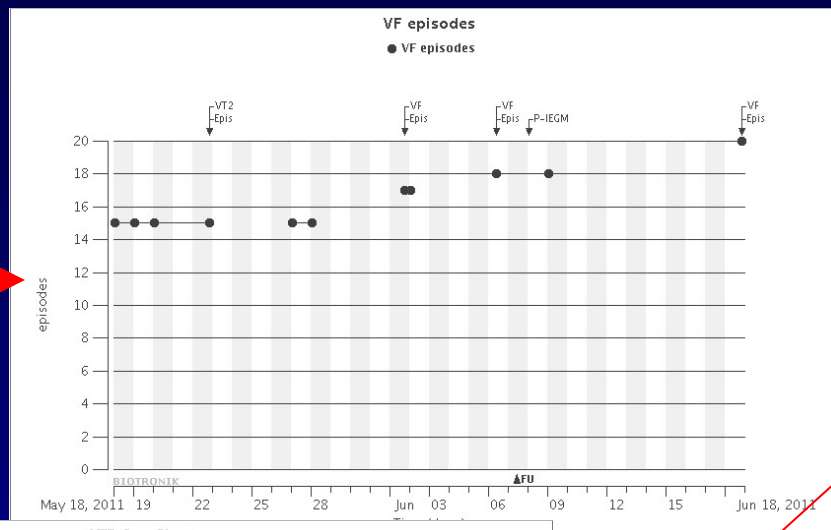
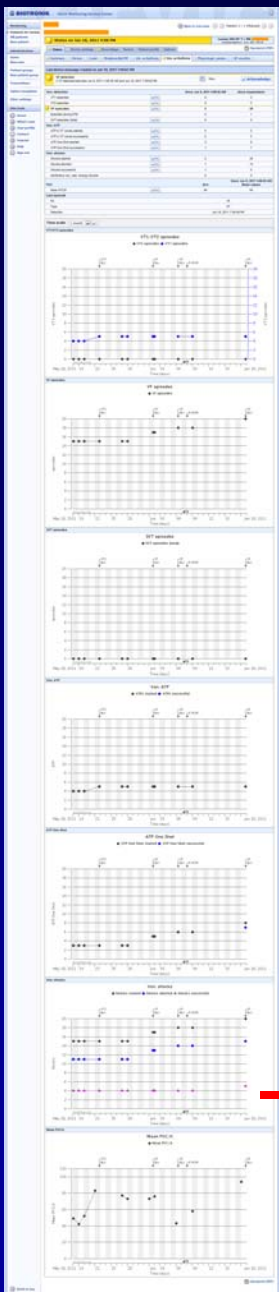
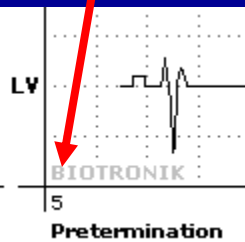
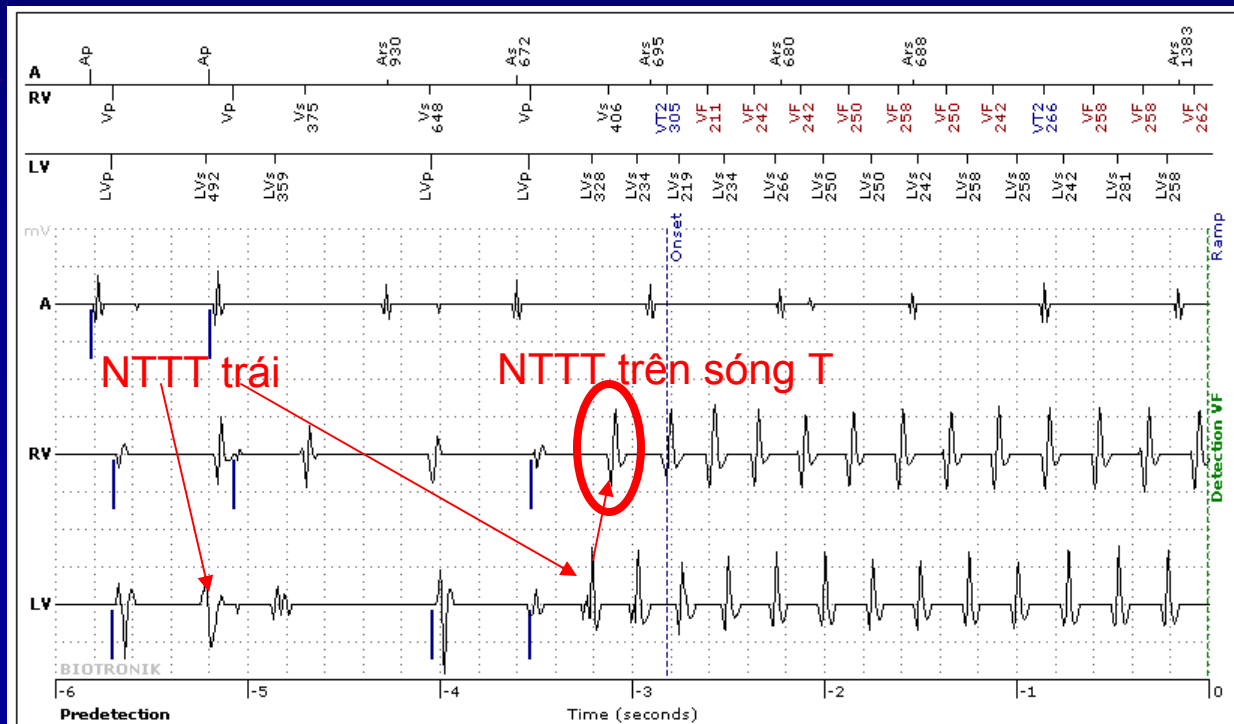
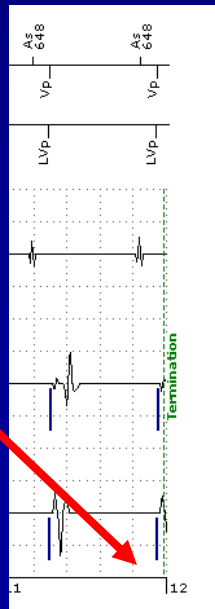
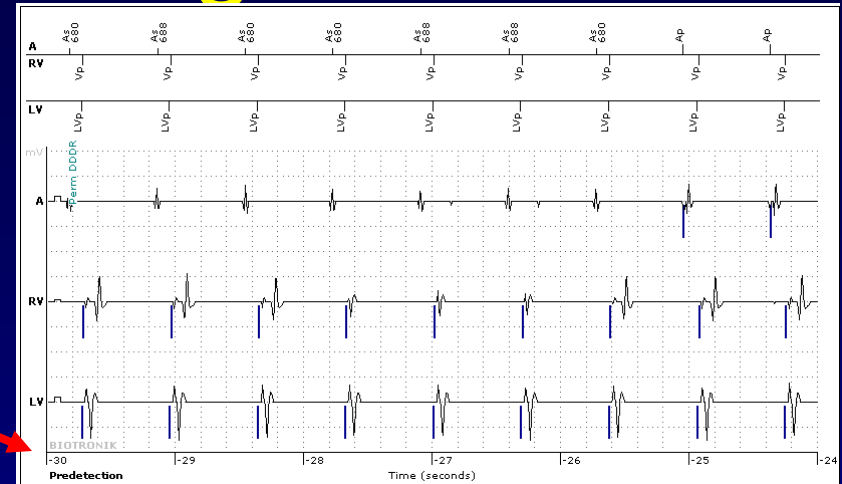
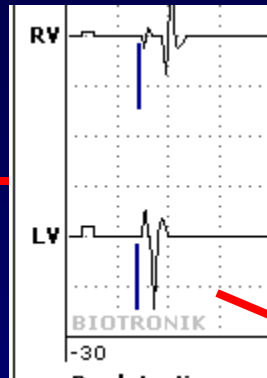
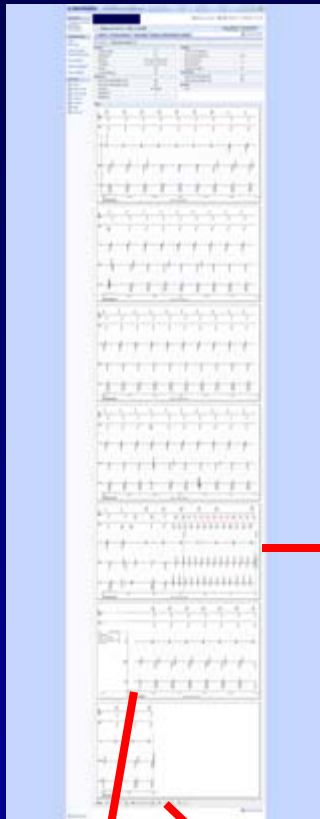
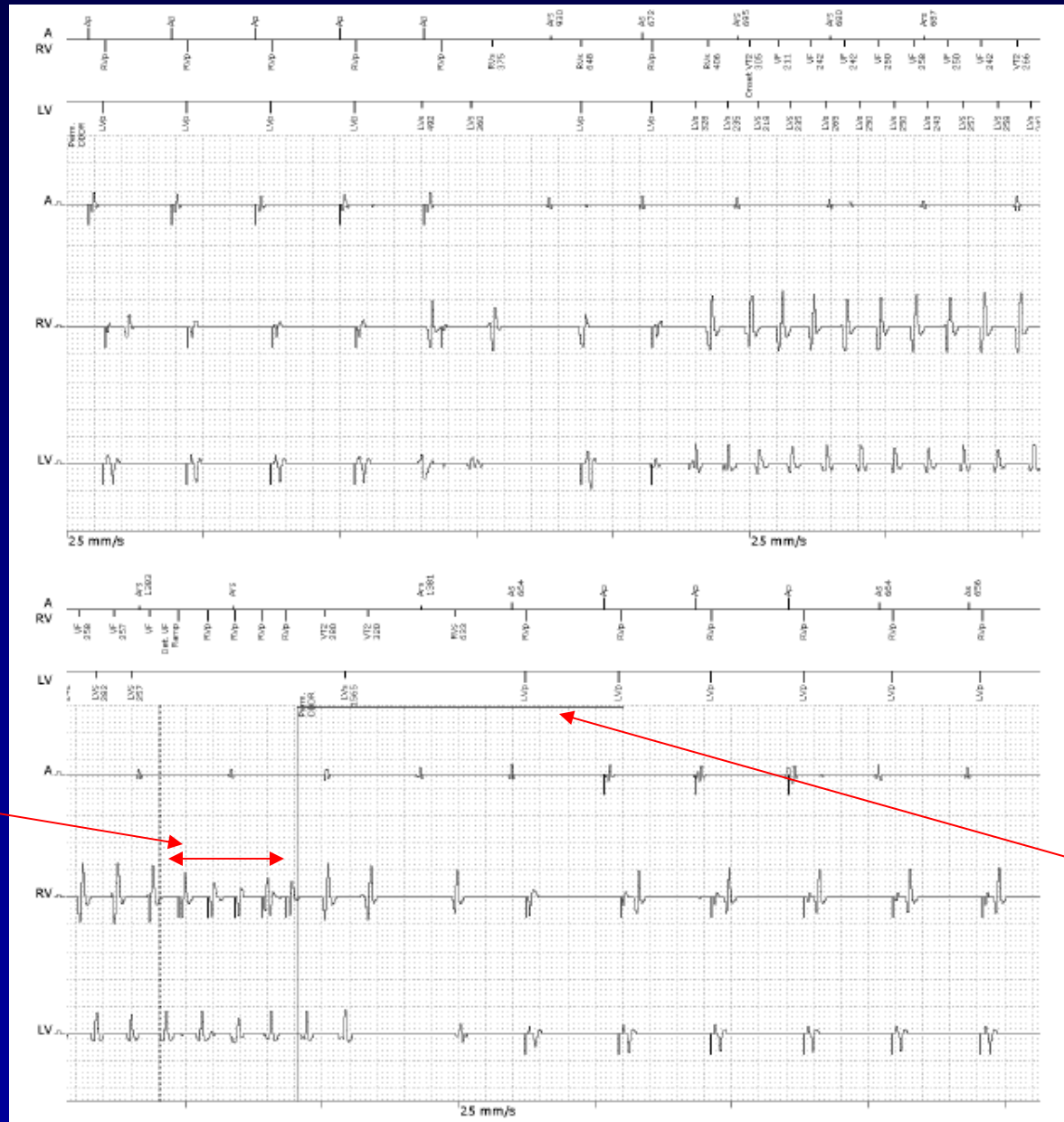
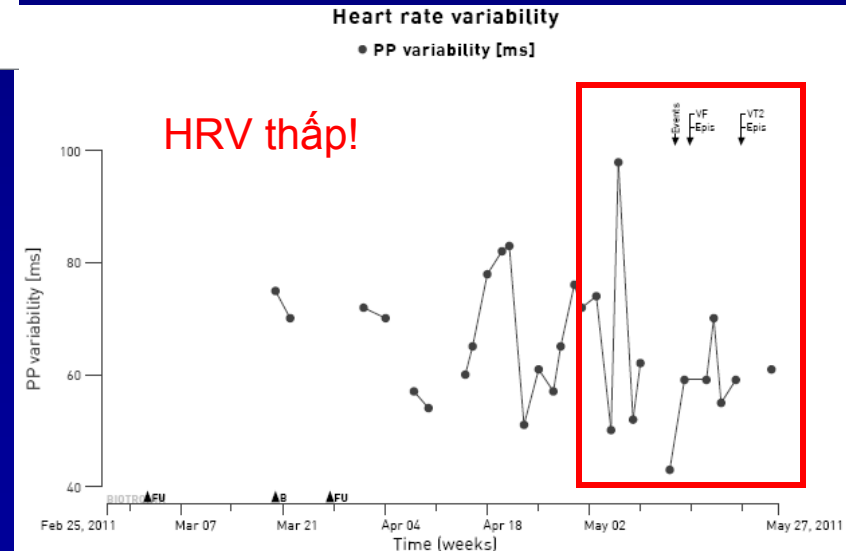
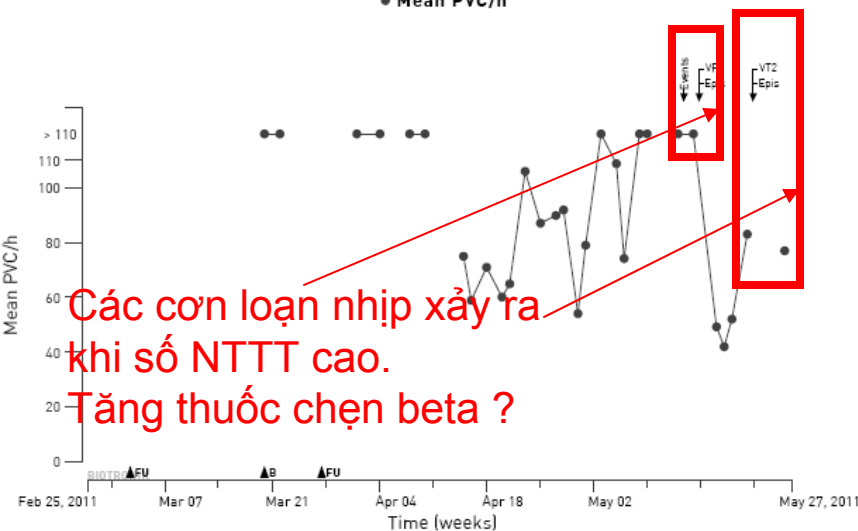
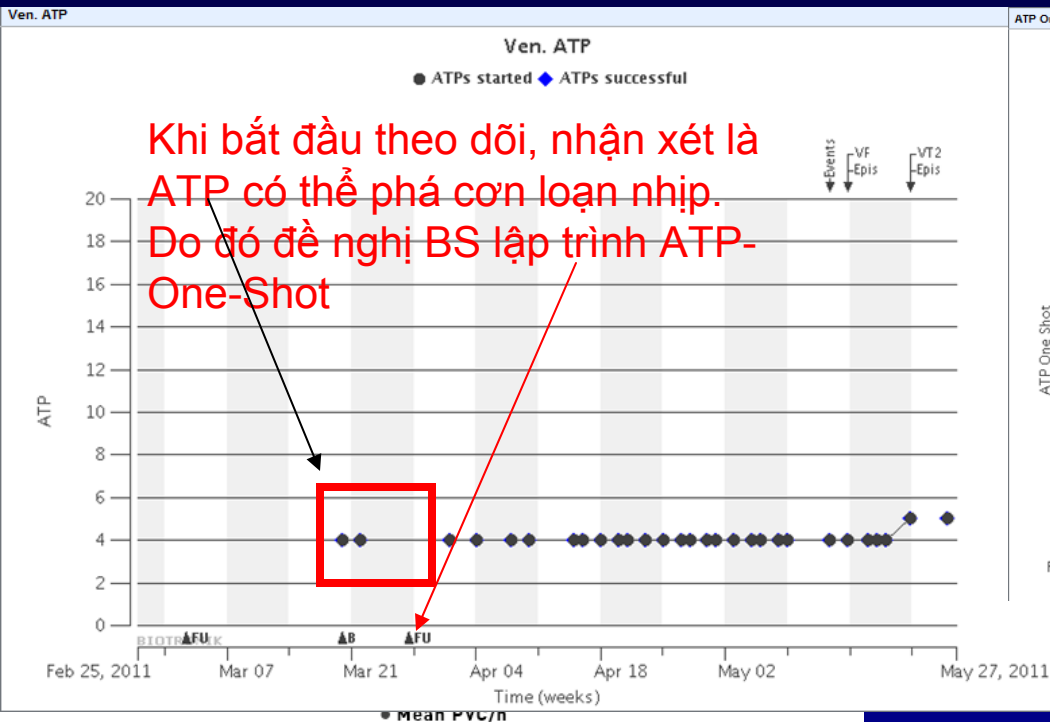
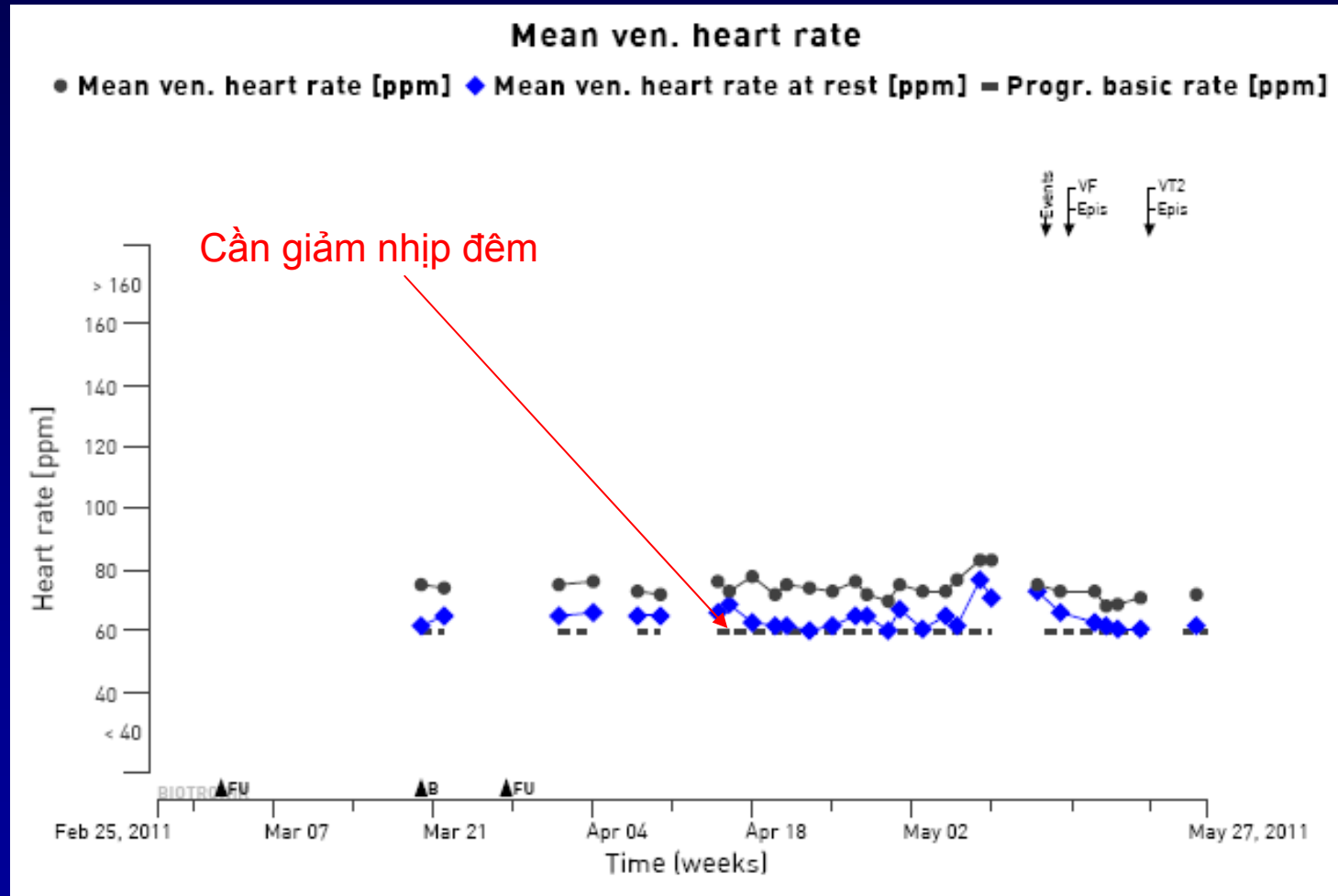
BIOTRONIK Home Monitoring is the only company with FDA & CE mark approval extensions

DEPARTMENT OF HEALTH & HUMAN SERVICES

Public Health Service

Food and Drug Administration
9200 Corporate Boulevard

- As a replacement of device interrogation during in-office follow-ups
- To provide early detection of arrhythmias, also silent, asymptomatic ones, enabling for earlier intervention

followups of BIOTRONIK implantable devices in many patients. Home Monitoring data is helpful in determining the need for additional in-office follow-up.

- * BIOTRONIK Home Monitoring—patients—who are followed remotely with office visits when needed—have been shown to have similar numbers of strokes, invasive procedures and deaths as patients followed with conventional in-office follow-ups.
- * BIOTRONIK Home Monitoring provides early detection of arrhythmias.
- * BIOTRONIK Home Monitoring provides early detection of silent, asymptomatic

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